MagLamp4Vision

Instructions for use


Dansk på mod-satte side

Unpacking and installation

Note: The arm in your MagLamp4Vision contains quite strong springs: When unpacking it, be sure to grab it firmly with both hands to avoid damage or injury.

Mount the table clamp onto the edge of a table, making sure to tighten the clamp's screw firmly. Then set the lamp into the hole atop the clamp.


Plug the wall adapter into a vacant mains outlet, and be sure to turn on the outlet switch: Now your MagLamp4Vision is ready for use.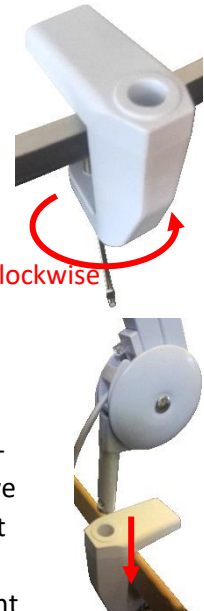

Counter-clockwise to tighten


On/off and dimming

Your MagLamp4Vision is controlled using the button above the lamp's lens: By pressing it one or more times, you can choose between four different light intensities, and "off" as illustrated at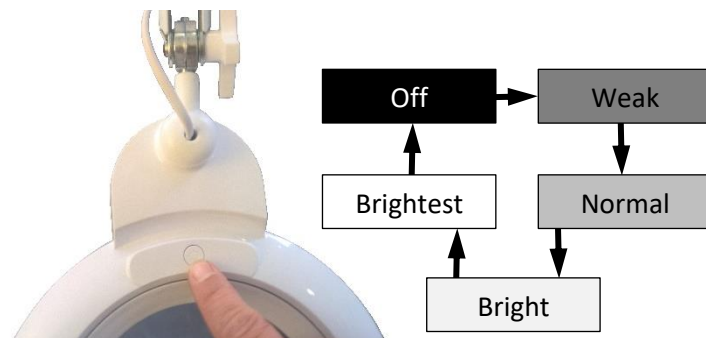

left.

Changing the Lens

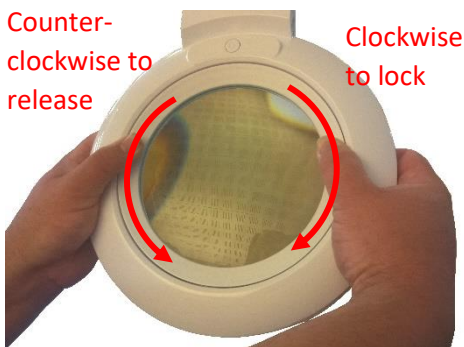to turn the lens rim clockwise, until it locks in place.

Your MagLamp4Vision ships with two lenses; 3×/8D and 5×/12D. To detach the lens, grab the lamp head with both hands, then use your thumbs to turn the rim of the lens counter-clockwise. Push the lens up from underneath the lamp head.

Note: When freed from the weight of the lens, the lamp head will tend to swing upwards.

Insert the new lens, and rotate it until it "drops into place". Use your thumbs as described above

Cloth pouch

The included cloth pouch must be fitted onto the lamp head, when the lamp is not in use: **This prevents sunshine from reaching the lens, getting focused onto e.g. paper and starting a fire!**

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